



THE ART OF FEELING

LET INNOVATION RELEASE THE JOYS OF LIFE



Unique glycosaminoglycan (GAG) combination therapy with a physico-mechanical action to help restore the GAG layer¹⁰



Caring Innovation

ialuril® contains glycosaminoglycans (GAGs) designed to support your bladder lining

Glycosaminoglycans (GAGs):

Naturally occurring molecules in the body; highly water-attracted and play key roles in maintaining the structural integrity and hydration of connective tissues like tendons, ligaments and bladders. Hyaluronic acid (HA) and chondroitin sulphate (CS) are a part of the GAG group.²

These GAG components attach and can bind to water; this means they can form a gel-like layer to protect the inside of the bladder.³

ialuril® contains:⁴

Hyaluronic acid
800mg/50ml



Binds to the damaged GAG layer barrier and restores it by protecting the bladder lining

Calcium chloride
440mg/50ml

Reduces the thickness of the solution, making easier to instill as well as allowing it to have a better ability to stick to the bladder lining

Chondroitin sulphate
1000mg/50ml

Improves bladder cell condition through reducing inflammation

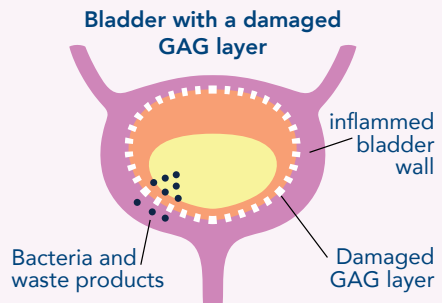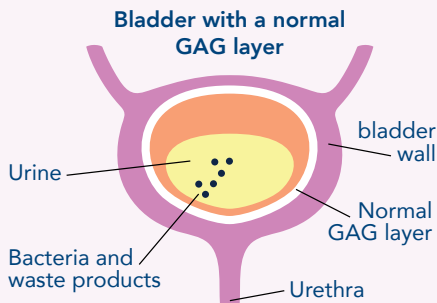

ialuril®: Unique combination GAG therapy



No animal-derived ingredients ¹



Pain Relief ^{6, 7, 8}



A unique combination GAG therapy containing Hyaluronic acid, chondroitin sulphate and calcium chloride ¹



Improved quality of life ^{6, 7, 8}

ialuril® Prefill



Well-tolerated ^{6,7,9}



Directly delivered into bladder for most effective use ¹



Symptom Relief ^{6,8,9}




Can self-administer ¹

How does ialuril work?


1 A healthy bladder has a good protective barrier

The inner lining (GAG layer) of the bladder functions as a barrier protecting the bladder wall from bacteria and irritants found in urine ⁵

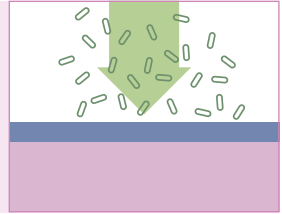


2 If the GAG layer becomes damaged, the bladder wall is unprotected

This can cause irritation, inflammation and damage, resulting in pain. ⁸

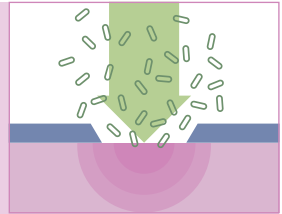

3 ialuril® replenishes the GAG layer

ialuril contains 2 important components that make up the GAG layer: HA and CS. ³ This enables it to effectively replenish the damaged GAG layer. ¹ Therefore, symptoms may be alleviated. ^{8,9}

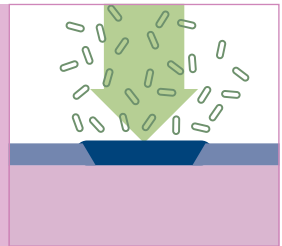

Recommended number of instillations¹

1 instillation

a week in the
1st month



1 instillation

every 2 weeks in
the 2nd month



1 instillation

once monthly in the **following months** until stable remission of the symptoms, or as prescribed by your doctor



Please note:

Administration should be under the direction of your Healthcare Professional who will decide the number of intravesical instillations based on your condition.

Administration of ialuril®:

ialuril® is instilled directly into the bladder either by a **catheter** or **via the ialuadapter** for catheter-free administration.¹

Via a catheter

- Any residual urine is drained away
- ialuril® is passed through the catheter into your bladder
- The catheter is then removed, leaving the solution in your bladder¹




Via the ialuadapter®

- You will need to pass urine immediately prior to treatment
- The ialuadapter® tip is inserted into the urethra
- ialuril is instilled into the bladder through the ialuadapter®¹



ONE OF THE
**WORLD
LEADERS**
IN HYALURONIC
ACID-BASED PRODUCTS

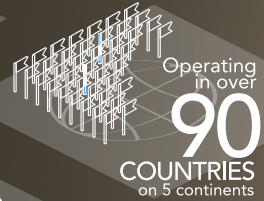

Operating
in over
90
COUNTRIES
on 5 continents



90
**exclusive
patents**
AND SEVERAL OTHERS
IN THE PIPELINE

The Art of Feeling

We develop solutions designed to support intimate and urological health, helping people reconnect with sensation, confidence, and closeness.

REFERENCES

1. IBSA. 2021.
2. Sodhi H, et al. Biomolecules. 2021;11(1):29.
3. Khastgir J. 2020.
4. Stellavato A, et al. PLoS One. 2019;14(6):e0218475.
5. Neugent ML, et al. ACS Infect Dis. 2023;9(4):1022–32.
6. Cicione A, et al. Can Urol Assoc J. 2014;8(9-10):E721-7
7. Cervigni M, et al. Int Urogynecol J. 2008;19(7):943–7.
8. Pichler R, et al. J Clin Med. 2024;13(7):2031.
9. Gacci M, et al. BMC Urol. 2015;15(1):46.
10. Brambilla L, et al. Med devices (Auckl). 2024; 17:47-58

IBSA UK

Sullivan House, 4-6 Colonial Business Park,
Colonial Way, Watford WD24 4PR, UK
Phone: +44 1923 233 466

