



PEROVIAL®

# THE ART OF FEELING

LET INNOVATION LEAD STRAIGHT BACK TO LIFE

The first HA-based treatment  
for the management of acute  
phase perionie's disease



Caring Innovation

# PEROVIAL<sup>®</sup>

## efficacious treatment for acute Peyronie's disease (PD)

During this phase you may experience...

- Lumps or plaques in 1 or more sides of the penis
- Pain during sexual intercourse or an erection
- Penile curvature

Perovial may be suitable for you as part of your PD management. Please speak to your healthcare professional to discuss your options.



Hyaluronic acid sodium salt  
for intralesional penile injection

# Hyaluronic acid (HA) aids PD



Tissue hydration and integrity<sup>3</sup>



Antioxidant<sup>2</sup>



Regulation of tissue responses during repair<sup>4</sup>



Wound healing<sup>5</sup>




HA plays an important role in maintaining moisture, flexibility and protection in tissues by attracting and holding water



It acts as a cushion and lubricant, helping protect cells from damage due to physical stress

# Perovial works by softening the plaque

1

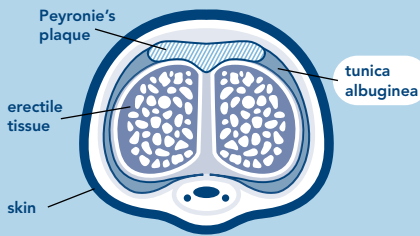


## A fibrous plaque develops in the tunica albuginea<sup>1</sup>

In the acute phase of PD, the plaque is being developed but not yet calcified, causing the penile curvature.

2

## Plaque is softened by HA<sup>1</sup>

The hyaluronic acid (HA) in Perovial has the ability to retain a high amount of water, allowing the softening of plaque to occur. It helps the correct healing process and stops or slows the progression of scarring.

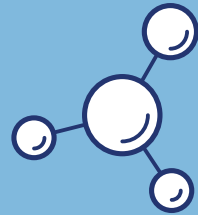

3

## Symptoms may improve

As the plaque softens, potential improvements include



Decrease in penile curvature<sup>3\*</sup>



Improvement in erectile function<sup>3,7#</sup>



Regression in plaque size<sup>3,7#</sup>



Improvement in penile pain and patient satisfaction<sup>3#</sup>



Good safety profile and tolerability<sup>3,6,7,8</sup>

1 injection per week




# What to expect<sup>1</sup>

Perovial is an **intrapenile treatment**.

**1**

The syringe and needle will be prepared by your healthcare professional (HCP)


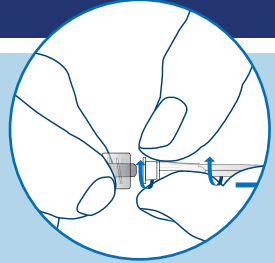


**2**

Your genital skin will be disinfected

**3**

Your HCP will locate the plaque by palpation

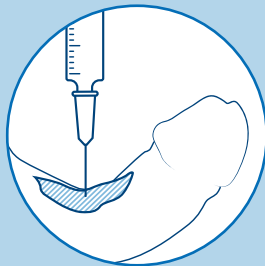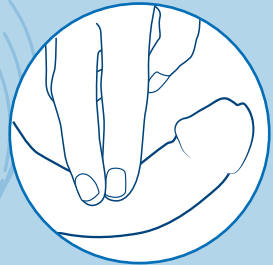

**4**

The plaque will be infiltrated firmly with the needle and the solution will be injected

*The volume of the injection will depend on the size of the plaque*

**5**

The area will be massaged after injection





ONE OF THE  
**WORLD  
LEADERS**  
IN HYALURONIC  
ACID-BASED PRODUCTS

Operating  
in over  
**90**  
**COUNTRIES**  
on 5 continents



**90**  
**exclusive  
patents**  
AND SEVERAL OTHERS  
IN THE PIPELINE

## The Art of Feeling

We develop solutions designed to support intimate and urological health, helping people reconnect with sensation, confidence, and closeness.

### REFERENCES

1. IBSA (2025). Perovial - Instructions For Use.
2. Capece M, et al. Uro. 2023;3:160-7
3. Favilla V, et al. Andrology. 2017;5(4):771-5.
4. Garantziotis S, Savani RC. Matrix Biol. 2019;78-79:1-10.
5. Iaconisi GN, et al. Int J Mol Sci. 2023;24(12):10296.
6. Cocci A, et al. World J Mens Health. 2021;39(2):352-7.
7. Zucchi A, et al. Sex Med. 2016;4(2):e83-8.
8. Cai T, et al. World J Mens Health. 2021;39(3):526-32.

\*Vs Verapamil #Vs baseline

### IBSA UK

Sullivan House, 4-6 Colonial Business Park,  
Colonial Way, Watford WD24 4PR, UK  
Phone: +44 1923 233 466

